



1 Harrier Court  
 Airfield Business Park  
 Elvington  
 York YO41 4AU  
 United Kingdom  
 Tel: +44 (0) 1904 607600  
 Fax: +44 (0) 1904 607601  
 Web: www.vet-way.com

## Biotect (PPE)

Product	Code	Case Quantity	Total Weight (kg)
Biotect PPE	PPE001	1	0.4

### Components Supplied

- Nitrile disposable Gloves
- Coverall
- Alcohol Hand Rub
- Disposable carry bag
- FFP3 Face Mask
- Safety Goggles
- 9kg UN3291 Clinical Waste bag
- Bioguard disinfectant wipes
- Overshoes

### Features

- Hand Rub has clip to attach to clothes or pocket for easy usage
- Convenient and disposable carry bag to carry all PPE to site
- Coverall is fully elasticised
- Face mask offers highest level of protection (FFP3)
- Safety goggles have head strap and air vents
- Bioguard wipes are individually foil wrapped

The Biotect, Personal Protection Equipment (PPE) was designed on the information provided on the CDC (Centers for Disease Control and Prevention) website and their 'Interim Guidance for Protection of Persons Involved in Avian Influenza Outbreak Disease Control and Eradication Activities'.

The kit provides all the necessary equipment required for the safety of personnel during the sample collection stage of the process.

The kit includes disposable nitrile gloves, which should be removed promptly after use and before touching non-contaminated items and environmental surfaces. The coveralls should be worn as an outer garment over clothing and the overshoes worn over footwear. The safety goggles are to protect the mucous membranes of eyes and the disposable respirator with a high level of respiratory protection (FFP3) should be placed over the mouth area.

All PPE provided is disposable and should be properly discarded of, by placing in the disposable carry bag and then in the Clinical Waste Bag provided. The Clinical Waste Bag should then be disposed of appropriately. Hand hygiene measures should be performed with the alcohol hand rub and Bioguard wipes after removal of all PPE.

